**BROCAS GATES E LARGOS
GATES AND PEESO BURS
FRESAS GATES Y LARGO**

ENGLISH

Indication:

Dental drills for endodontics are conventional rotary instruments, to be used as aids in the chemical-mechanical preparation of root canals, that is, used for enlargement and regularization of the upper portion of the canal (cervical and middle third) and removal of the dental structure (removal of dentin concrescence opposite to the cingulate and dentin elbows), in addition they also act in the removal of gutta percha in cases of retreatment or defilling for cementation of pins.

Intended user:

Duly qualified dental professionals.

Target population:

Patients undergoing endodontic procedures that involve chemical-mechanical preparation of root canals, retreatment or partial defilling for cementation of intraradicular pins. The product can be used in patients of different age groups, according to clinical indication and judgment of the professional in charge.

Fundamentals of its operation and action:

Dental drills for endodontics have an active tip with a flame shape and diameter that obey an ascending order, with the numbering from 1 to 6, having a total length of 28mm and 32mm and can penetrate up to 19mm inside the canal, when this is possible and desirable. The instrument number is indicated on the instrument by grooves at its base.

Contraindication:

- Do not use with inadequate speed, pressure or angulation, which may lead to fracture of the drill or excessive removal of the tooth structure.
- Do not use in deep, narrow or hard-to-reach areas, where the instrument may become locked or cause structural damage.
- Do not use in procedures that require direct instrumentation inside the root canal, especially in apical areas.

Instructions for use:

- An inspection of the components for functionality and cleanliness is recommended before use. Before submitting the product to use, STERILIZATION is mandatory;
- The long tooth axis must be strictly observed, being aware of possible existing inclinations;
- The root canal should be completely flooded with irrigating solution before receiving the instrument and should be copiously irrigated and aspirated after use;
- The rotary instrument must enter the channel loosely, working freely;
- The rotary instrument must enter and exit the root canal, and can never stop inside the canal. This prevents it from locking and having its removal difficult or its fracture from occurring;
- Preference should always be given to the use of rotary instruments that do not have a cut at the end of their active tip;
- When using these instruments, they should always be removed from the contra-angles of the micromotor to avoid sharp puncture accidents with contaminated biological material;

Form of presentation

- Non-sterile product;

Select the dental drill for endodontics that will be used and check the RPM according to the table below:

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Manual Cleaning Procedure:

- Wash with the help of a soft brush and enzymatic detergent under running water;
- Submerge the product in a specific cleaning solution for surgical instruments (follow the manufacturer's instructions);
- Remove from solution;
- Care should be taken to clean cracks and other hard-to-reach areas. Visually inspect to confirm debris removal. Repeat the cycle if necessary.
- Rinse the ends thoroughly under running water until they are visibly clean.
- Pat the ends dry with clean compressed air or paper towels.

Ultrasonic Cleaning Procedure:

For ultrasonic cleaning, it is recommended to proceed as follows:

- Prepare a neutral pH cleaning solution (enzymatic detergent) following the agent manufacturer's instructions for correct concentration, exposure time, temperature, and water quality.
- Place the dental tips in the appropriate location for the ultrasonic equipment, following the manufacturer's instructions for use.
- Completely immerse the device in the cleaning solution and program the equipment for at least fifteen (15) minutes.
- Rinse dental drills thoroughly under running water until they are visibly clean.
- Dry the drill bits with clean compressed air or paper towels.

Sterilization Requirements:

- Pack in material suitable for sterilization;

BROCAS GATES E LARGOS
GATES AND PEESO BURS
FRESAS GATES Y LARGO

• Sterilize in an Autoclave at 121°C for 20 minutes or sterilization parameters should be followed according to the relevant policies/regulations of the country where the device is being used.

Warnings and Precautions:

- Non-sterile product
- The sterilization of dental drills for endodontics is not replaced by cleaning.
- Once sterilized, dental drills for endodontics should be opened only during operation under aseptic conditions.
- Exclusive product for dental use.
- Do not use the product if the packaging is damaged, opened or wet.
- During the procedure, the professional must use all recommended Personal Protective Equipment (PPE), including goggles. It is also recommended to wear protective glasses on the patient.
- Only dental surgeons should use this product, as they have the scientific information necessary for its proper use.
- LOT Number: It is important to keep the lot number, identified on the product label, to be informed in any occurrences with the manufacturer.

Conservation and storage:

- The product must be in an airy, dry place and away from light in order to maintain ideal storage and transport conditions, as well as its physical and chemical integrity.
- Keep the products in their original packaging until the moment of use.
- The carrier must be informed of the contents.
- The care to be taken in transport is demonstrated in the packaging, thus ensuring protection to the instruments from shipment to delivery to the customer.
- Packaging should not be left to the elements.

Discard:

The disposal of the material must follow the relevant local legislation.

Shelf Life:

Not applicable. The product is manufactured from non-perishable materials and supplied non-sterile. When stored under the conditions indicated, it retains its properties indefinitely.

Life:

5 uses (5 reuses), however, it is of paramount importance that before each use a check is carried out, that is, if the dental polishers are in perfect condition for use.

Additional information:

- If necessary, 3R provides instructions for use in printed form, at no additional cost, upon request by email (administrativo@3r.ind.br) or by phone: (+55 11-5525-0590 / +55 0800-773-5525), and we will get back to you within a maximum of 7 business days from the request.
- Always check the website: www.microdont.com.br, the most up-to-date version of the instructions for use corresponding to the product.

Composition:

Dental Drills for Endodontics are Manufactured from AISI 303 Stainless Steel Stainless Steel - (AISI 303)

BROCAS GATES E LARGOS
GATES AND PEESE BURS
FRESAS GATES Y LARGO

ELEMENTS	COMPOSITION
Carbon (C)	0,15 Máx.
Manganese (Mn)	2,00 Máx.
Silicon (Si)	1,00 Máx.
Phosphorus (P)	0,20 Máx.
Sulphur (S)	0,15 Máx.
Chromium (Cr)	17,00 – 19,00
Nickel (Ni)	08,00 – 10,00

Symbology:

Symbol	Symbol Title	Description
	Validity	Indicates the date after which the health product should not be used.
	Non-sterile	Indicates a health product that has not undergone a sterilization process.
	Medical device	Indicates that it is a medical device.
	Recyclable packaging	The packaging is recyclable.
	CE Mark	It indicates that the medical device is in compliance with Regulation (EU) 2017/745 (MDR).
	See instructions for use	Indicates the user's need to refer to the instructions for use.
	Date of manufacture	Indicates the date the health product was manufactured.
	Keep out of sunlight	Indicates a health product that needs protection from light sources.
	Maker	Indicates the manufacturer of the health product as defined in Directive 90/385/EEC, 93/42/EEC and 98/79/EC of the European Community.
	Lot Code	Indicates the manufacturer's batch code so that the batch or batch can be identified.
	Catalog Number	Indicates the manufacturer's reference number so that the health product can be identified.
	Authorised representative in the European Community	Indicates the authorized representative in the European Community.
	For professional use only	It indicates that the medical device must be used exclusively by a qualified health professional.

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