BRUSH - APLICADORES DESCARTÁVEIS
BRUSH - DISPOSABLE MICRO APPLICATORS
BRUSH - MICROAPLICADORES DESECHABLES

ENGLISH

Indication:

Disposable Micro Applicators are indicated for the controlled application of substances in dental procedures. They can be used for the deposition of evidencing solutions (caries, bacterial plaque, among others), adhesive agents, sealants, eugenol, hemostatic solutions, fluorides and materials used in teeth whitening techniques, even helping in the movement of the gel during the procedure. The folding rod facilitates access to different regions of the oral cavity and favors precision in application.

Intended user:

Duly qualified dental professionals.

Target population:

Patients undergoing dental procedures that involve topical application of solutions, agents or fluid materials, regardless of age or clinical condition, as indicated and judged by the professional in charge.

Plea in law:

Its function is based on the controlled transfer of the material by means of a flexible tip with fibers that retain small amounts of solution by surface tension, without significant absorption, allowing precise dosing and avoiding waste.

Contraindications:

- Do not use in patients with known hypersensitivity to the components of the device.
- Do not use with highly caustic substances or strong solvents that may compromise the integrity of the fibers or the stem.
- Do not use for the application of high viscosity or very thick materials that may deform or damage the fibers of the microapplicator.

Instructions for use:

- Remove an Applicator from the packaging.
- Touch the active part of the solution to be applied.
- Apply the solution in the region of interest.
- The professional can choose the type of applicator (medium/blue, fine/green and extra fine/lilac), according to the size of the region to be worked and the amount of product to be applied.

Form of presentation

- Non-sterile product;
- Single use.
- Package containing 100 units.
- Available in three sizes: regular, fine and extra-thin.

Cleaning Procedure:

Not applicable. The product is for single use, so it should not be cleaned or reused.

Sterilization procedure

Not applicable. The product is supplied non-sterile and should not be sterilized. Dispose of dental waste after use in accordance with local regulations.

Warnings and Precautions:

BRUSH - APLICADORES DESCARTÁVEIS
BRUSH - DISPOSABLE MICRO APPLICATORS
BRUSH - MICROAPLICADORES DESECHABLES

- Product for professional use only.
- Keep out of reach of children.
- Do not subject the device to high temperatures.
- During operation, the professional must use all personal protective equipment, including goggles.
- Wear protective glasses on the patient.
- LOT Number: It is important to keep the lot number, identified on the product label, to be informed in any occurrences with the manufacturer.

Conservation and storage:

- Keep the product in its original packaging until the moment of use. Inspect the packaging before use.
- The product must be stored in an airy, dry place protected from light, in order to maintain ideal storage and transport conditions, as well as its physical and chemical integrity.

Discard:

- The disposal of the material must follow the relevant local legislation.

Shelf Life:

Not applicable. The product is manufactured from non-perishable materials and supplied non-sterile. When stored under the conditions indicated, it retains its properties indefinitely.

Life:

Not applicable. The product is for single use only and should be discarded after use.

Additional information:

- If necessary, 3R provides instructions for use in printed form, at no additional charge, upon request by email (administrativo@3r.ind.br) or by phone: (+55 11-5525-0590 / +55 0800-773-5525), and we will get back to you no later than 7 business days from the request.
- Always check the website: www.microdont.com.br, the most up-to-date version of the instructions for use corresponding to the product.

Composition:

Polypropylene and polyamide fibers.

Symbology:

Symbol	Symbol Title	Description
	Validity	Indicates the date after which the health product should not be used.
	Non-sterile	Indicates a health product that has not undergone a sterilization process.
	Do not reuse	Indicates an intended health product for use or for use on a single patient during a single procedure.
	Keep dry	Indicates a health product that needs to be protected against moisture.
	Medical device	Indicates that it is a medical device.
	CE Mark	It indicates that the medical device is in compliance with Regulation (EU) 2017/745 (MDR).

BRUSH - APLICADORES DESCARTÁVEIS
BRUSH - DISPOSABLE MICRO APPLICATORS
BRUSH - MICROAPLICADORES DESECHABLES

	Date of manufacture	Indicates the date the health product was manufactured.
	Keep out of sunlight	Indicates a health product that needs protection from light sources.
	Maker	Indicates the manufacturer of the health product as defined in Directive 90/385/EEC, 93/42/EEC and 98/79/EC of the European Community.
LOT	Lot Code	Indicates the manufacturer's batch code so that the batch or batch can be identified.
REF	Catalog Number	Indicates the manufacturer's reference number so that the health product can be identified.
EU REP	Authorised representative in the European Community	Indicates the authorized representative in the European Community.

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